



**ASSUT EUROPE**  
SUTURE CHIRURGICHE



*More than surgery*

# Covid-19

## IgG/IgM RAPID TEST



### ACCURANCE

High-purity antibody with high accuracy




### EFFICIENT

Test result is available in 15 mins




### CONVENIENT

Suitable for finger blood

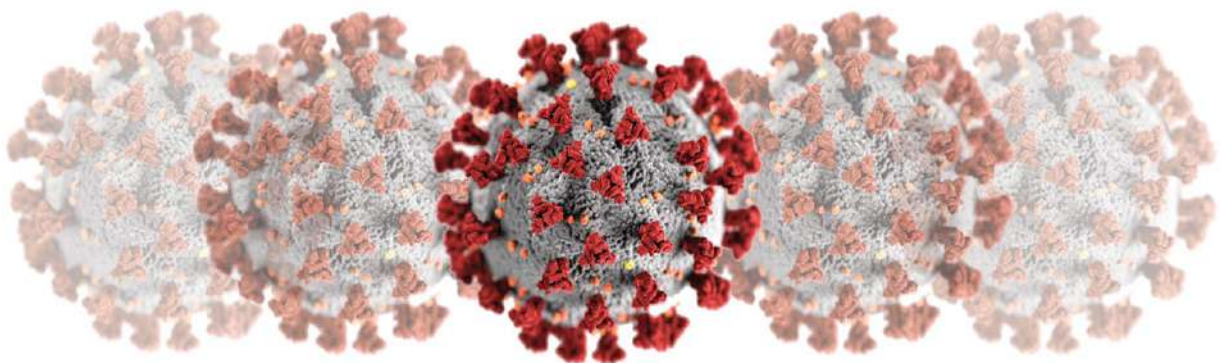


# Covid-19

## IgG/IgM RAPID TEST

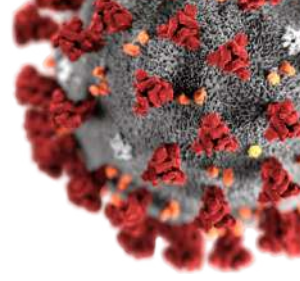

COVID-19 IgG/IgM Rapid Test Cassette is a lateral flow chromatographic immunoassay for the qualitative detection of antibodies (IgG and IgM) to Novel coronavirus in human Whole Blood/Serum/Plasma.

It provides an aid in the diagnosis of infection with Novel coronavirus.

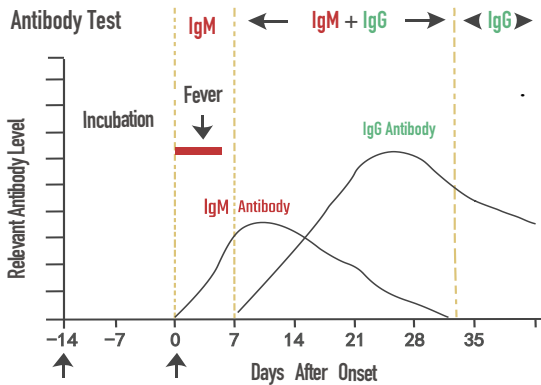

Antibodies will be secreted after virus invasion. Immunoglobulin M (IgM) comes out first, acting as the early sign of infection. Immunoglobulin G (IgG) comes out later, arising a more specific and stronger reaction against the virus.

### Operation Instruction: Correct use, convenient e quick



1. Disinfect the fingertip with a alcohol prep pad



2. Use a sterile lancet to take fingertip blood



3. Collect the blood with a dropper



4. Add 1 drop of blood into the sample well



5. Slowly add 2 drops of sample diluent into the sample well




6. Interpret the result in 15 mins



7 e 8. The waste after use should be treated as infectious materials, and do not be discarded randomly. We suggest to use EC STER SPRAY



### Result Interpretation

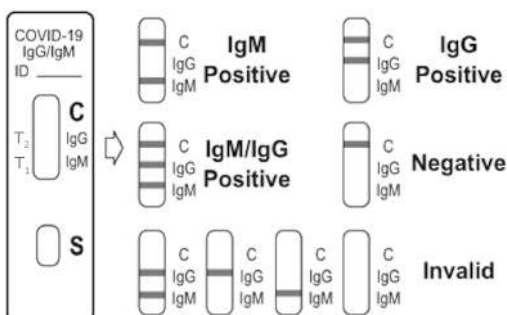

CODE	DESCRIPTION	BOX
COVID-19	IgG/IgM Rapid Test	Packs 25
5230/26	EC STER SPRAY	10 single-dose vials
5230/15	EC STER SPRAY	15 single-dose vials